
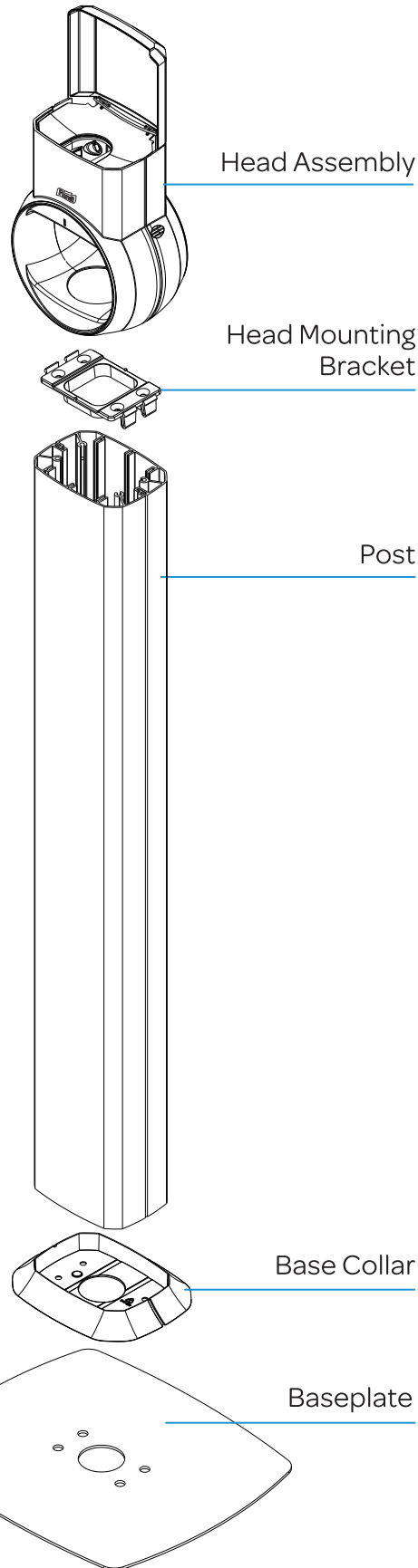
PURELL® FS6 & FS8 Dispenser Stand Assembly Instructions

For video instructions, go to www.PURELLSOLUTION.com/FS-Stand

Package Contents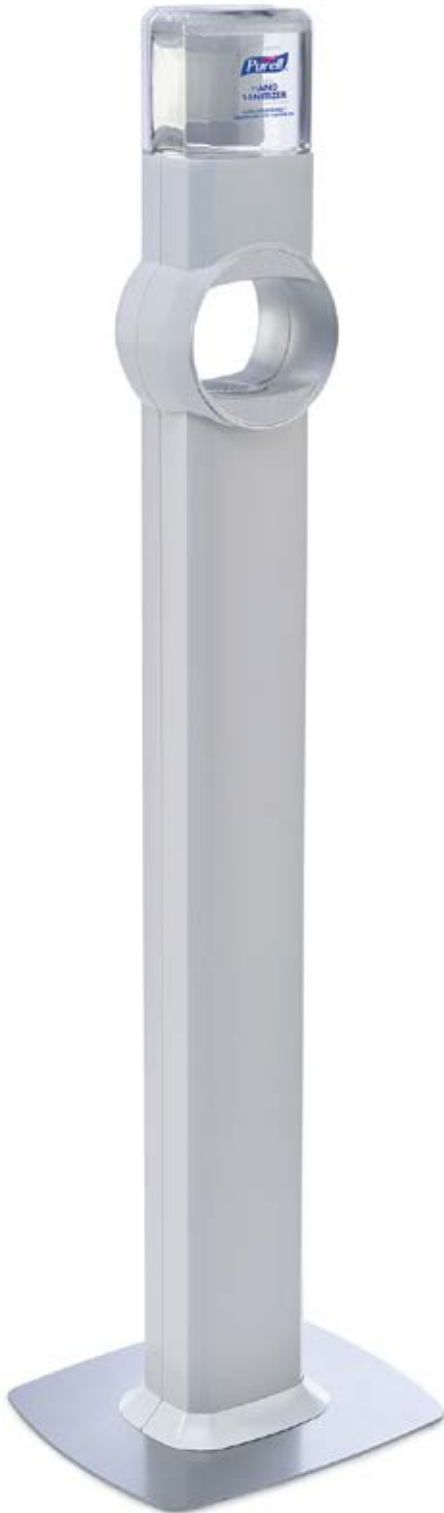


Strainer


Drip Tray

Screws (8)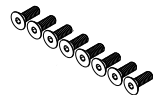


Wrench
Hex Wrench

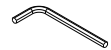ASSEMBLY INSTRUCTIONS


REFER TO FIGURE A

1. Slide the **base collar** over one end of the **post** (post ends are identical; base collar will attach to either end).
2. Set the **baseplate** on top of the **post** (note feet are pointing up) and **base collar**. Align the four (4) holes.
3. Use the hex wrench to secure with four (4) screws.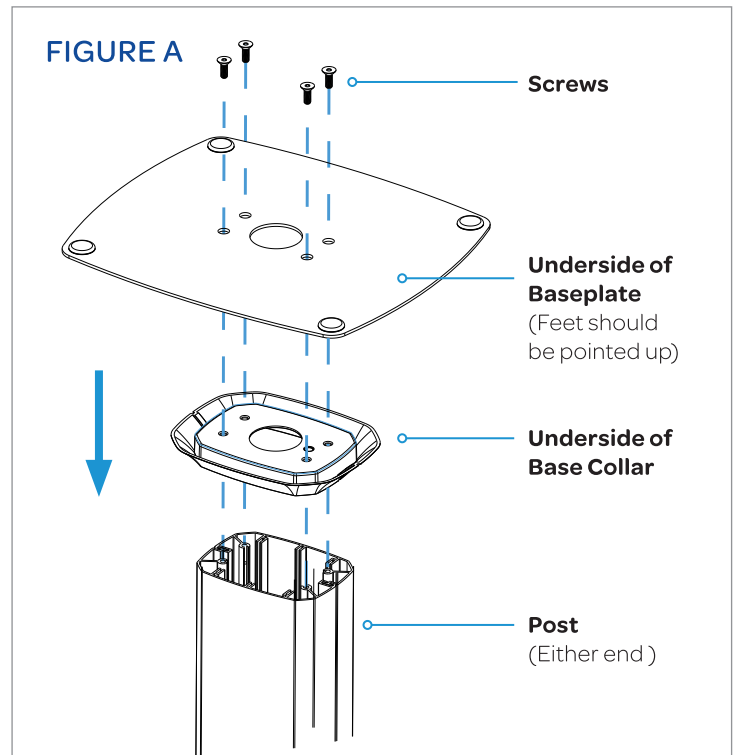

REFER TO FIGURE B

4. Turn assembly over so the **baseplate** is resting on the floor.
5. Position the **Head Mounting Bracket** on the free end of the **post** and use the hex wrench to secure with four (4) remaining screws.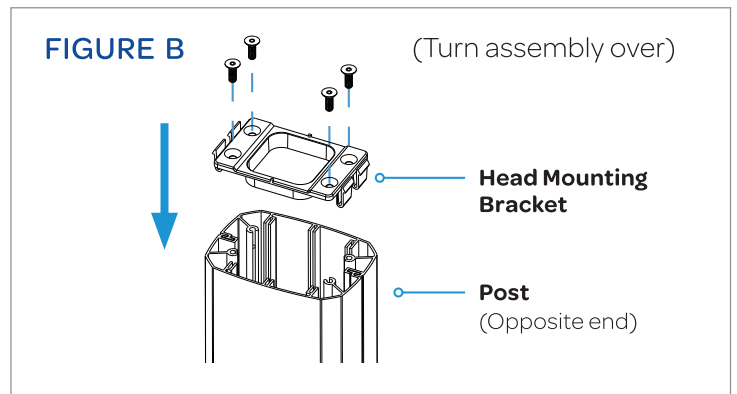


REFER TO FIGURE C

6. If closed, open head assembly from side.
7. Align the head subassembly over the end of the post; inserting the tab *in a downward motion* into the slot of the post.
8. Push the head subassembly down and insure all four (4) clip features of the mounting cap are seated over the ribs.

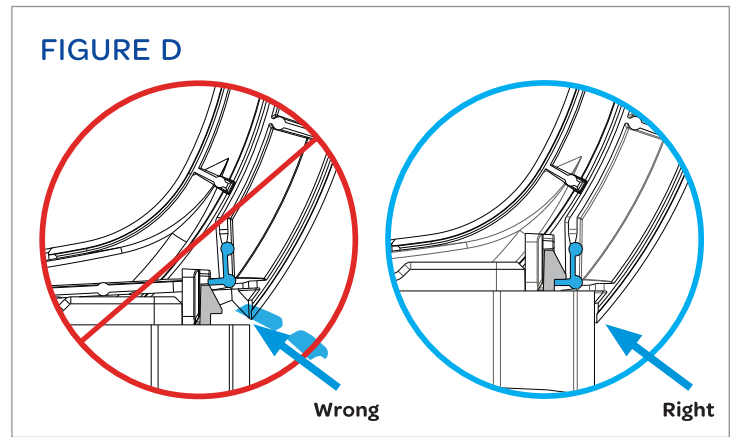
Head will click into place when secure.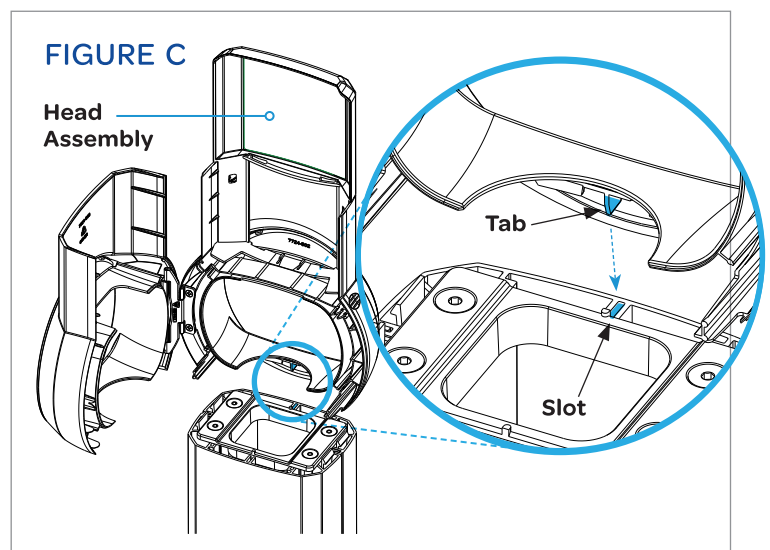

ASSEMBLY INSTRUCTIONS


REFER TO FIGURE D

 Please ensure the long rib is positioned under the clip on BOTH sides before proceeding to the next step. If INCORRECT – Push down on the rib until fully seated in the CORRECT position. Gently pull up on head to verify that it is fully latched in place.

REFER TO FIGURE E


9. Nest the strainer in the drip tray. Orient with the **tabs** front and back as shown, and place the pair in the recess at the top of the post.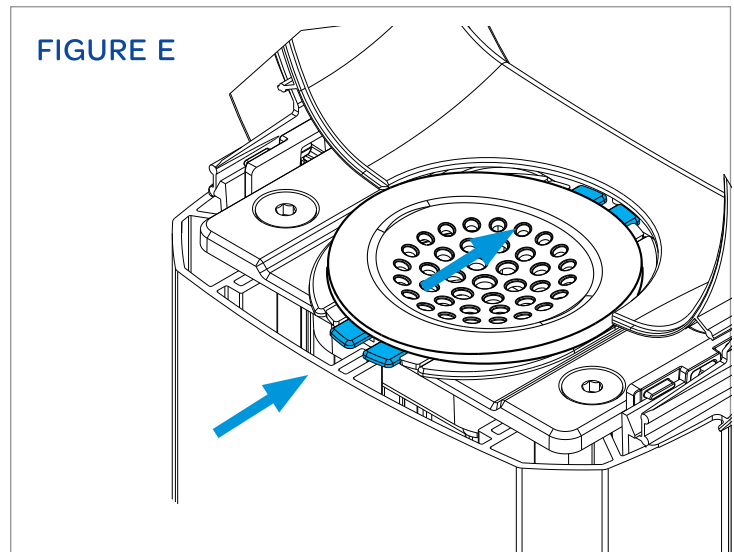

STAND REFILL REPLACEMENT

REFER TO FIGURE F

1. Open the front cover of the stand.
 - a. Push out on the inside ridges (A) and in on the outside divot (B) to unhook the latch.
 - b. Pull the cover forward and rotate to open.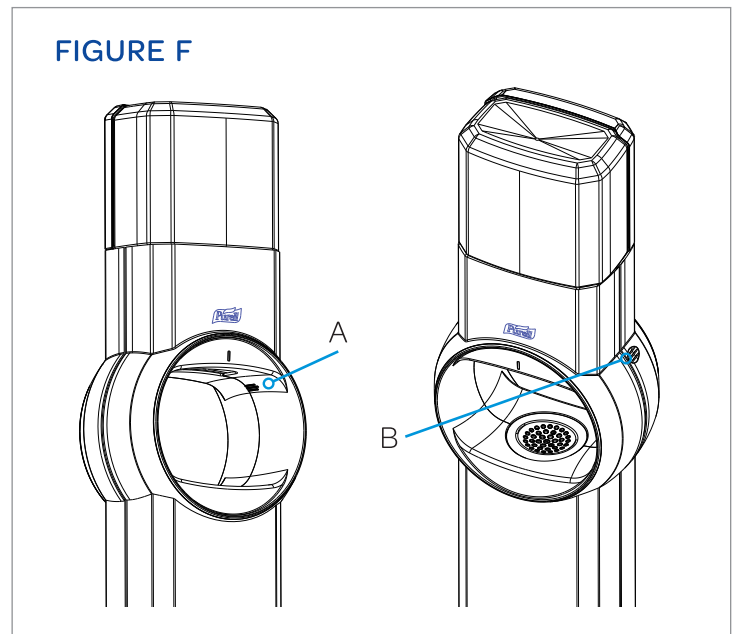

PURELL® FS6 & FS8 Dispenser Stand Assembly Instructions


STAND REFILL REPLACEMENT

REFER TO FIGURE G

1. Remove the empty bottle.
 - a. Push the green release.
(It may be on opposite side.)
 - b. Pull the bottle forward.
2. Insert a full bottle of PURELL Sanitizer, pressing until the bottle snaps into position.
3. Close the stand cover and squeeze both sides of the head to ensure that it latches.

For video instructions, go to
www.PURELLSOLUTION.com/FS-Stand

FIGURE G

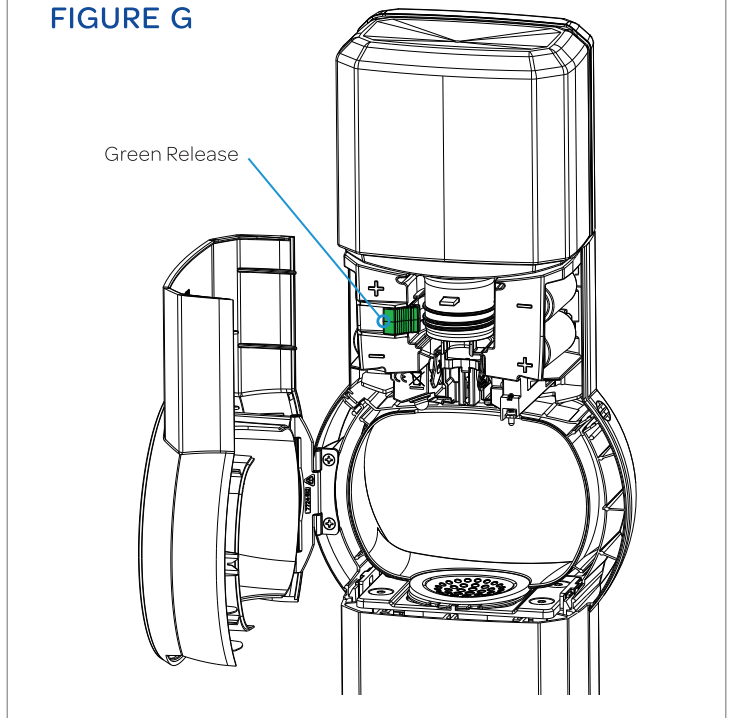WARNINGS & TROUBLESHOOTING

Dispenser Status LED

FS6 & FS8

Green Flash - every 7 seconds = good

FS8 Only

Red Pulse + no dispense + charge level low

Red flash - every 7 seconds = out of service

- Wrong refill
- Unrecoverable clog

Red/Green alternating flashes = system fault (replace)

Troubleshooting

No Dispensing, Motor Does Not Run

- Hands are not directly beneath sensor
- No refill is loaded
- Incorrect refill is loaded

No Dispensing, Motor Runs

- Refill is empty or loaded incorrectly
- Refill pump is not primed

Cleaning and Maintenance

Clean exterior surfaces of the dispenser using warm water and mild soap. DO NOT USE ABRASIVE CLEANERS. Dry with a soft cloth.

Rated 6Vdc

Battery Operated Appliance

Classification, per UL 60601-1

Internally Powered

Type B Applied Part

IXPO

Not suitable for use in the presence of a flammable anesthetics

Warnings

Use dispenser only as directed. Do not use in areas where dispenser may be exposed to wet environments such as showers or outdoors. Do not install or use in extreme hot or cold locations. Dispensed product intended for external use only. Refer to refill for actions in case of ingestion.

Definitions of Symbols

Type B Applied Part

Attention, consult accompanying documents

MEDICAL EQUIPMENT

WITH RESPECT TO ELECTRIC SHOCK,
FIRE AND MECHANICAL HAZARDS ONLY
IN ACCORDANCE WITH UL60601-1.IEC60601-1.
CAN/CSA C22.2 No. 601.1

©2018. GOJO Industries, Inc. All rights reserved.

GOJO Industries, Inc.
One GOJO Plaza, Suite 500
P.O. Box 991 • Akron, OH 44309-0991
Tel: 1-330-255-6000 • Toll-free: 1-800-321-9647
Fax: 1-800-FAX-GOJO

Visit www.PURELLSOLUTION.com
to learn more.

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©2018. GOJO Industries, Inc. Tous droits réservés.